

Livfria®

Urine collection devices
**designed
for you.**



Clinisupplies




Designed for you.

We understand how continence challenges can affect your daily life. It is our purpose to help you **live more freely.**

We have designed the **Livfria**® Leg and Night bags around you, to be more discreet and secure.

Live more freely.

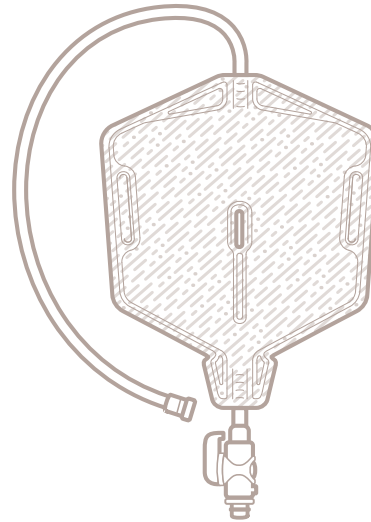

Livfria® Discreet
Opaque Leg Bag



Traditional Clear
Leg Bag

Discreet and Secure.



Designed to be discreet

Livfria® leg and night bags are more discreet than traditional leg and night bags. You can choose to no longer see your urine and our products are shaped to be less visible under clothing.



Designed to be secure

Livfria® has a unique connector that when clicked into place, maintains a closed-link system up to 4 times longer than traditional leg and night bags.

28
days

Closed-link system
using Livfria®
Click-Close Connector




Click-Close



How to use Livfria® step by step.

1. How to connect your Livfria® Click-Close Connector to your urinary catheter.

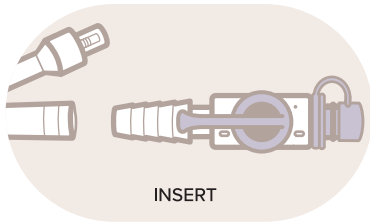

Make sure you wash your hands and wear gloves provided when connecting **Livfria®** leg or night bag to the connector.




Insert the **Livfria®** connector to the urinary catheter.

28
days


**This connector can remain
in place for 28 days.**

After 28 days, you will need to change the connector, refer to the instructions for use.

2. How to connect your Livfria® Leg Bag to your Livfria® Click-Close Connector.

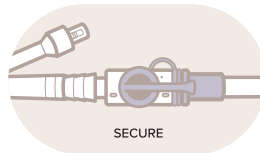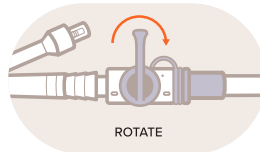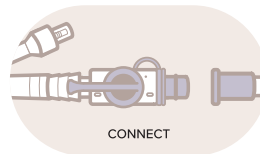

Connect your leg bag tubing to the the connector. Rotate the connector tap 180 degrees until it is pointing towards your leg bag, your system is now securely closed.

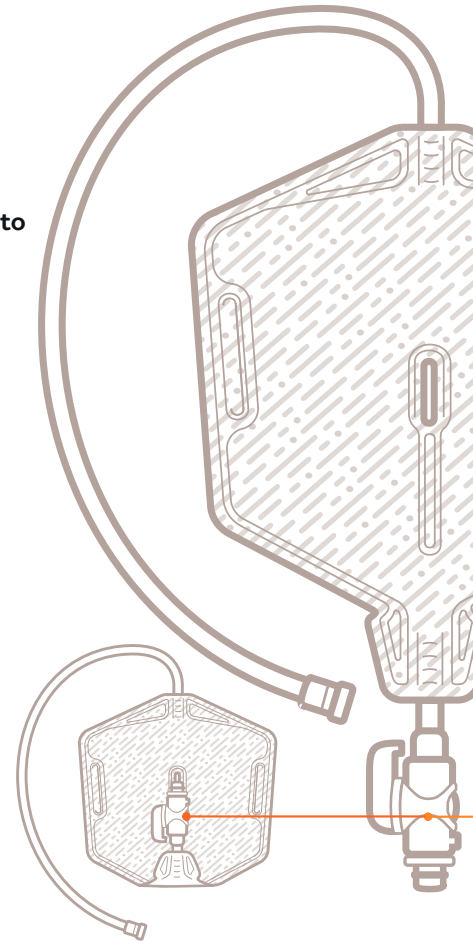



Livfria® bags all feature free-flow tubing to avoid obstruction of flow.

The unique pattern on our **Livfria®** leg bag is designed to be discreet.

Livfria® leg bags are chambered for improved comfort.


The connector tap on the bottom of the leg bag can be folded back on itself for additional discretion. The tap can be held in place by the small tab on the leg bag.

7
days

The leg bag can remain in place for 7 days




3. How to connect your Livfria® Night Bag.

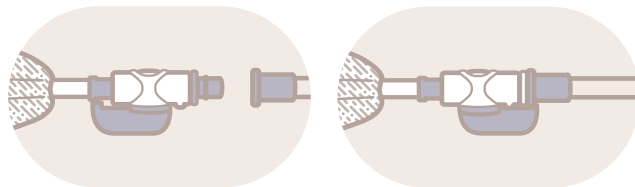

If you are using a night bag for additional overnight urine collection, connect the night bag to the click-close connector tap on the leg bag.

Once connected, turn the tap 180 degrees so that it points to the night bag instead of the leg bag. The urine will now pass through the leg bag and drain into the night bag.

The **Livfria®** night bag can be attached in the evening and detached in the morning by simply turning the tap 180 degrees so that it points to the leg bag. The urine will now only collect in the leg bag. The night bag is now safe to remove and empty.

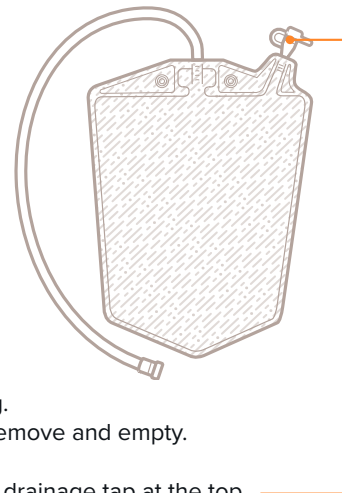

Empty the night bag using the drainage tap at the top.

Livfria® Leg Bags

with Click-Close Connector

Leg bags come with:

- 10 Livfria® Leg Bags
- 2 Livfria® Click-Close Connectors
- 1 Livfria® Leg Bag Strap

Available in **discreet** opaque
or **semi-transparent** styles.

Volume	Tube Length	Bag Style	Order Code
Regular	Short	Discreet	LFLB-DRS
Regular	Long	Discreet	LFLB-DRL
Large	Short	Discreet	LFLB-DLS
Large	Long	Discreet	LFLB-DLL
Regular	Short	Semi Transparent	LFLB-SRS
Regular	Long	Semi Transparent	LFLB-SRL

Leg and Night Bags

Livfria® Leg Bags

are available in short or long tube depending on your best fit.


Livfria® Leg Bags

can be both discreet opaque or semi-transparent depending on your specific requirements.




Click-Close Connector

connects Livfria® devices together for security.

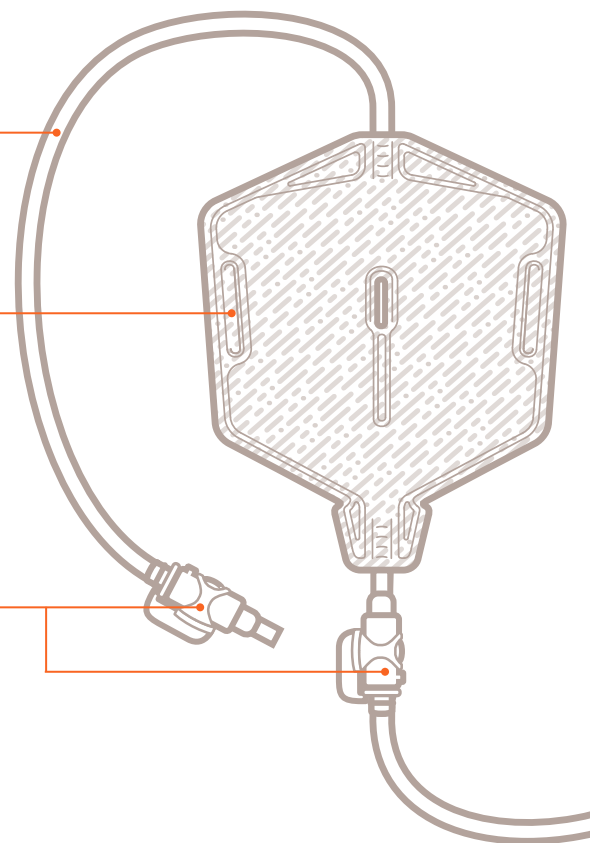

designed for you.

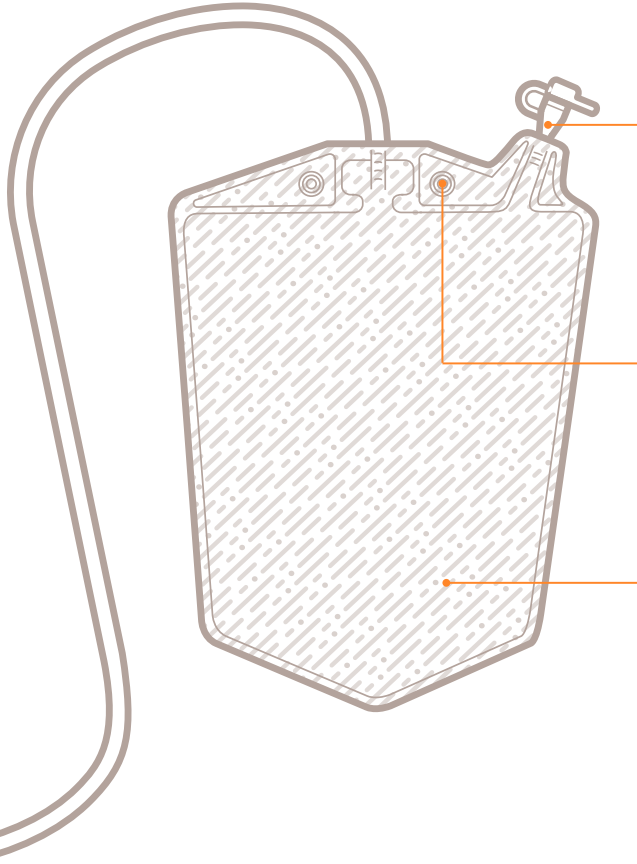

Livfria® Night Bags have the drainage tap at the top, this keeps it away from the floor and makes it easier to empty.

Reinforced eyelets make it easy to hang or place the night bag on a stand.

Regular and Extra-large night bags are available dependent on your needs.

Livfria® Night Bags

Night bags come with:

- 10 Livfria® Night Bags
- 2 Livfria® Click-Close Connectors

Volume	Quantity	Order Code
Regular	10	LFNB-DR
X-Large	10	LFNB-DXL

or without connectors

Regular	10	LFNB-DR-NC
X-Large	10	LFNB-DXL-NC

Livfria® Leg Straps

Volume	Quantity	Order Code
Regular	5	LFLS-R



Nightingale

Delivering for you.

Your Home pack contains 7 days supply:


- 1 x Livfria® Click-Close Connector
- 1 x Livfria® Discreet Leg Bag
- 1 x Livfria® Semi-Transparent Leg Bag
- 1 x Livfria® Night Bag
- 1 x Livfria® Leg Bag Strap

If you would like to order more **Livfria®** supplies, please contact us and your delivery will arrive between 24-72 hours to ensure you never run out.


We offer a range of complimentary items that help you manage your routine. These items can be ordered and delivered to your home address with your product orders.

Order online or by phone:

www.nightingaledelivery.co.uk
0800 304 7434



Our superior customer service team is here by your side



Fast, accurate ordering with discreet home delivery



Delivering all your bladder and bowel products



With a dedicated team of product specialists and nurses to support you



Excellent*

★ Trustpilot

Nightingale home delivery is part of Clinisupplies, helping you to find and order the bladder or bowel products and accessories to live more freely.

*February 2024